# Okped® orione

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
**Fig.1:** Apply felt with the round shape around the affected part

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
**Fig.2:** Felt do not must cover the deformity

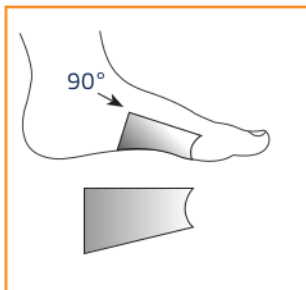SINISTRO / LEFT

DESTRO / RIGHT

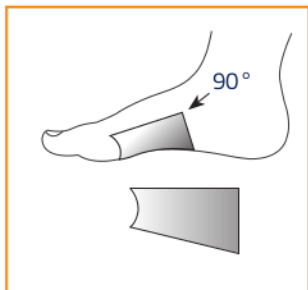SAFTE S.p.A.  
CERTIFIED  
ISO 9001  
Quality System

CE

SAFTE

SAFTE S.p.A. - Via Adamello 13 - 10093 Collegno (TO) Italy  
Tel. +39 011.4038111 Fax +39 011.4038126  
e-mail: info@safte.it www.safte-italia.com www.safte.it

# Okped® orione

FELTRO ADESIVO PER ALLUCE VALGO

ADHESIVE FELT PAD FOR HALLUX VALGUS

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ADHESIVE FELT PAD FOR HALLUX VALGUS  
THANKS TO THE PATENTED DESIGN, THE FELT CREATES A FREE AREA THAT REDUCES PRESSURE AND FRICTION OF THE SHOE AROUND PAINFUL AREA. EASY AND FAST APPLICATION

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## ADHESIVE FELT PAD FOR HALLUX VALGUS

Hallux valgus is the forefoot deformity of the great toe. If this occurs, the inflamed protrusion that constantly pushes against the shoe becomes painful.

Thanks to the special design and material, the OK PED felt creates a free area that reduces pressure and friction of the shoe around painful area.

- Relieves toe pain
- Improves immediate mobility and well-being
- Made of hypoallergenic and adhesive pre-cut felt
- Soft protection applied directly to skin
- Easy and fast application
- Reusable
- Available left and right version
- Fits into any walking shoe except dancing or wide court shoes

Stops discomfort caused by a misaligned big toe

Rediscovered foot mobility

Easy to put on

Removes pressure on the affected area and brings about instant relief and dramatically pain decreases.