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MAX user weight: 130 kg

INSTRUCTIONS: *Height adjustment*

The crutches must be adjusted according to the height of the user.

Canadian crutches have 10 holes where you can adjust the height.

- Unscrew the grey locking ring.
- Push down the metal locking pin so that it retracts completely into the pole.
- Select the desired height by adjusting the crutch with an upward or downward movement.
- Release the metal locking pin in front of the required hole position.
- Retighten the locking ring after choosing the desired hole height

CAUTION

Use the crutches with caution upon wet or slippery surfaces.

Before use, always make sure that each locking pin protrudes fully from the selected hole position and that the height is the same for both crutches

Avoid using in contacts with wounded limbs.

NEVER use damaged or modified crutches.

MAINTENANCE and CARE

Cleaning: Regular cleaning with neutral soap. DO NOT use bleach, alcohol or any solvents

Tip: Periodically check the rubber tips for rips, tears, cracks or worn tread; they should be replaced when necessary. Use original spare parts – Item 31

Warranty: Two years from proven purchase date. Except spare or wearable parts and forcing.

DEMARTA VIRGINIO S.r.l.
VIA Europa 1 Cerrione (BI- Italy)

01.07.ST01B
01.07.ST01N

Classe I

Ed. 22/02/2026

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Distributed by: SAFTE S.r.l - Via Adamello, 13 - 10093 Collegno (TO) - Italy